CALVING ATTACHMENT INSTRUCTIONS FOR USE





1. LOAD COW

- Walk cow into Wrangler Attach rump chain behind her
- Close headbail
- Attach belly straps (as required)

2. LOWER CALVING ATTACHMENT

- Swivel calving attachment down
- Slot **O** pipe inserts into slide bar ring and secure with 4mm R clip
- Repeat on other side



The calving attachment has two height/angle settings. Use top sidebar ring for level pull or use bottom slidebar ring for downwards pull

THE

3. PULL CALF

/RANGLER

- Rope attaches to pin on back leg winch, round back leg bar, then onto calf
- Tension using back winch & pressure on rope



LIMITED



PH 0800 466 837 www.thewrangler.co.nz